

SERIES 5 DENTAL CHAIR


OVERVIEW

MODELS:

C5550 Series 5 Dental Chair with 200 Watt Transformer

C5551 Series 5 Dental Chair with 200 Watt Transformer, 230V

C5552 Series 5 Dental Chair with 200 Watt Transformer and AirSoothe

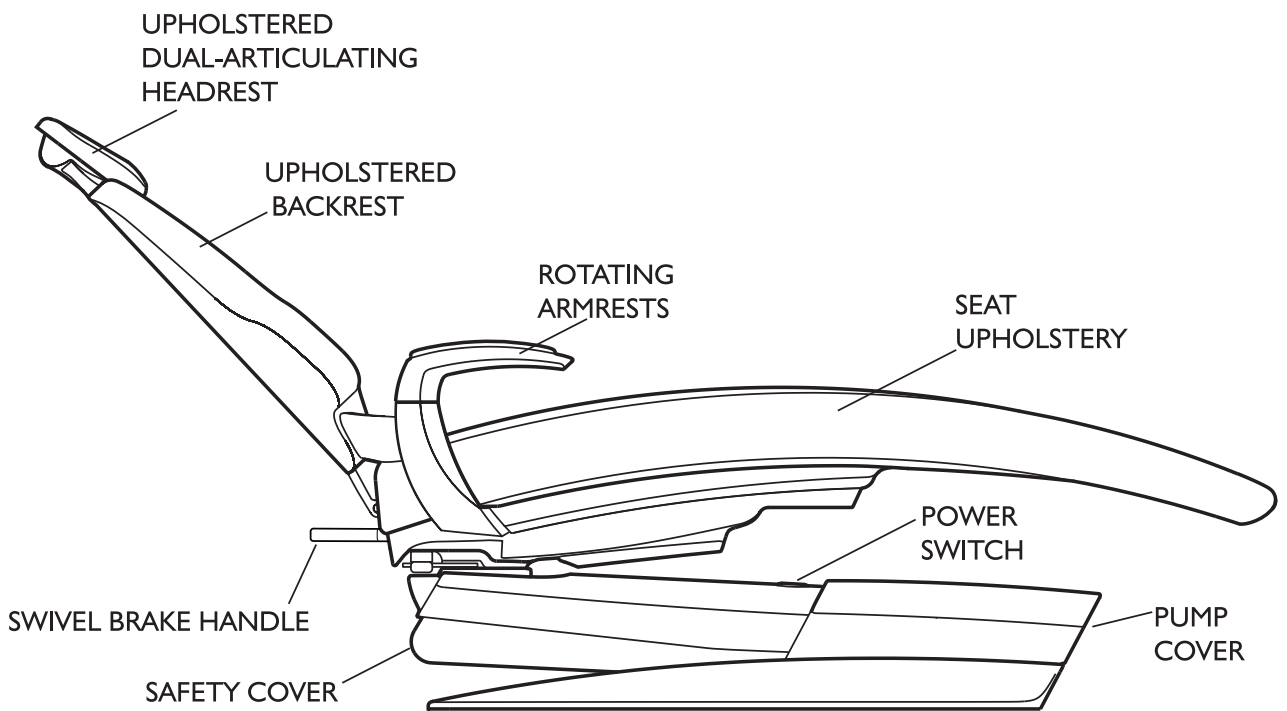

INTRODUCTION

SYMBOLS

The following symbols may be used throughout this product manual:



CAUTION: General Caution. Failure to carefully follow the described procedure may result in damage to the equipment.



WARNING: Failure to carefully follow the described procedure may result in damage to the equipment, the operator, and/or the operator.



ELECTRICAL HAZARD: Risk of electrical shock present. Ensure that power is disconnected before attempting this procedure.



NOTE: Take note of additional important information. Not a warning or caution.



Manufacturer



Manufacturing Date



Type BF Applied Part



Electrical Testing Lab



Power OFF



Power ON



Protective Earth (ground)



Alternating Current



Advisable to consult accompanying documents



Refer to instruction manual/booklet



Waste Electrical and Electronic Equipment



Conforms to applicable European Directives (Essential Requirements)



Conforms to applicable UK Directives (Essential Requirements)



Chair Function: directional movement key



Chair Function: Chair Preset position keys 0 and 1



Program key



Chair Function: Return to last position key



Unique Device Identifier



Medical device

TABLE OF CONTENTS

Overview	2
Introduction	3
General Information	4
General Safety	6
Operation - Foot Switch	7
Operation - Headrest	8
Operation - Swivel	9
Operation - Safety Stops	10
Optional Quick Connection Box	11
Optional Airsoothe™ Massage Technology	12
Maintenance	13
Electromagnetic Compatibility	15

GENERAL INFORMATION


INTENDED APPLICATION AND USE

The DCI Edge dental patient chair is a device intended to position the patient in such a manner to provide to the dental practitioner visual and manual access to the oral cavity. This device should provide for the ergonomic necessities of most patients, both adults and children, throughout its range of movement. The device shall also be the means of supporting delivery systems for the purpose of standard dental procedures.

The dental chair is classified as Class I device under rule FDA CFR 21 and a Class I device under EU/MDR 2017/745.

PRODUCT IDENTIFICATION

The DCI Edge Series 5 Patient Chair can be identified by the label on the front end of the seat casting. This label states the model number, serial number, electrical specifications, and manufacturing date.

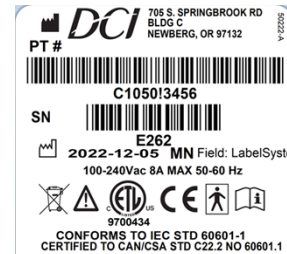

ELECTRICAL SPECIFICATIONS

115VAC	IEC Medical Classification I
8 Amps	TYPE: B Chair upholstery
50-60Hz	Operation: Continuous
	Duty Cycle: 20 Seconds ON, 5 Minutes OFF
	Splash Protection: IPX0 (FOOT SWITCH IPX1)
230VAC	
2.5 Amps	
50-60Hz	

ELECTRICAL DEVICE INTERFERENCE

To guarantee the operational safety of electromedical devices, it is recommended that the operation of mobile radio telephones in the medical practice or hospital be prohibited. Strong EMI sources such as electro surgery delivery systems or x-ray delivery systems may effect performance. If performance problems occur, move the chair to another electrical circuit or physical location.

TECHNICAL DOCUMENTATION

The manufacturer will make available upon request circuit diagrams, component part lists, descriptions, and calibration instructions to technical personnel responsible for the installation and service of this equipment.



NOTE: In the event of a serious incident, whether it involves injury, harm, or any unexpected adverse effects experienced while using our device, we kindly request that you immediately notify our company and the competent authority of the Member State in which the user and/or patient is established.

ENVIRONMENTAL CONDITIONS

Operating Conditions:
Temperature: 67-76° F
Humidity: 20-60%
Altitude: <5,000 m

Shipping conditions:
Temperature: -68 - 122° F
Humidity: 10-90%

INCOMPATIBLE DEVICES

For safety reasons, only DCI original devices and accessories approved for this product or accessories from third parties released by DCI may be used.

PRODUCT DISPOSAL

Contact your local, authorized dealer for disposal of this device to ensure compliance with your local environmental regulations.

GENERAL INFORMATION

PREVENTATIVE INSPECTION

The performance of the equipment can be affected by use over time. Inspect joints and tensioning screws as a regular maintenance item to ensure proper positioning of the device. See “**Maintenance**” section of this manual for detailed instructions.

PRE-USE REQUIREMENTS

All operators of this device must read this document in its entirety before use.

Inspect the chair for signs of wear or damage before use. Do not use the chair if any components appear loose, broken, or malfunctioning.

Ensure that all installation steps are completed by a trained technician as described in the Installation Instructions for this device. Only trained dental professionals should handle adjustments. Discontinue use immediately and contact a qualified technician if the chair exhibits any signs of electrical malfunction (for example, unusual noises, smells, or visible damage).

Ensure all functions and features as defined in the “**Operation**” sections of this manual are operating properly. Check and securely lock headrests and other support components before seating the patient. Avoid excessive force when adjusting to prevent damage or detachment. Verify that the patient is comfortably and securely positioned before making any adjustments. Pay special attention when seating pediatric patients or those with limited mobility to ensure their limbs are safely positioned.

Follow the instructions on cleaning and disinfecting in the “**Maintenance**” section of this manual before first time use of this device.

Ensure that all staff responsible for the maintenance, disinfection, and cleaning of this device are fully trained on the information provided in the “**Maintenance**” section of this manual.

CHAIR WEIGHT CAPACITY

Load testing has been conducted to over 4 times the below recommended load capacities without failure to ensure the proper performance and safety of your DCI chair.

The following loads should not be exceeded:

Patient Load: 550 LBS

Chair Mounted Accessory Load (ie System Light and Delivery): 150LBS

GENERAL SAFETY

WARNINGS & CAUTIONS



WARNING: This product must be disinfected before use.



CAUTION: Only authorized service technicians should attempt to service this equipment. Use of other than authorized technicians will void the warranty.



CAUTION: Use a licensed electrician for all wiring.



ELECTRICAL HAZARD: To avoid risk of electric shock, this equipment must be connected only to supply mains with protective ground.



WARNING: Modification of this equipment is not allowed.



WARNING: Failure to disinfect this device between patients could expose the user and patient to cross contamination and bioburden/biocontamination.



WARNING: Power cords and their associated parts cannot be substituted without increased risk of shock or fire. Use authorized replacement parts only. Power cords must be installed by qualified personnel. Ensure all service loops, strain reliefs and cord guards are in place and that line, neutral and ground wires are secured.



WARNING: Do not exceed the weight limits for patient and chair mounted delivery (p.5). Doing so may cause personal injury and/or damage to the chair.



WARNING: To avoid possible injury and/or damage to the chair, do not apply full body weight on the headrest, backrest, toeboard and armrest. Doing so may cause the chair to tip.



WARNING: To isolate from mains power, unplug the cord. Make sure to leave the mains cord accessible.



WARNING: Press any key on footswitch or accessory control keypad to terminate motion of the chair while it is moving.



WARNING: Do not connect extension cords or power strips to the electrical outlet underneath the seat upholstery.



WARNING: Do not allow children unsupervised access to the dental chair and auxiliary equipment.



NOTE: This product is intended for use by trained dental/medical professional only.

OPERATION - FOOT SWITCH


CONTROLS

FOOT PAD

- **BACKREST INCLINE:** Raise the backrest by pressing your foot toward the right of the Foot Pad.
- **BACKREST RECLINE:** Recline the backrest by pressing your foot toward the left of the Foot Pad.
- **CHAIR UP:** Raise the seat of the chair by pressing your foot toward the top of the Foot Pad.
- **CHAIR DOWN:** Lower the seat of the chair by pressing your foot toward the bottom of the Foot Pad.

KEYS

- **RETURN:** Press this key to return the chair to the previous position (the last position held for longer than 10 seconds). Press any key to cancel while in motion.
- **PRESET POSITIONS:** Press “0” or “1” to activate preset positions. Press any key to cancel while in motion.

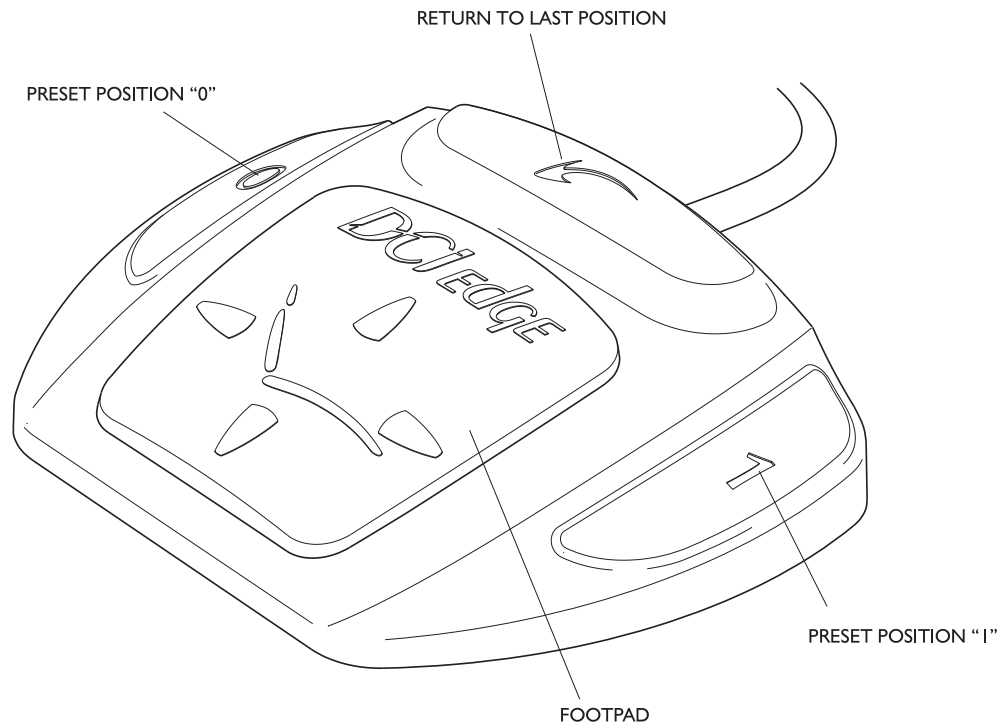

SETTING PRESET POSITIONS

ASSIGN POSITION TO “0” OR “1”


1 Adjust the position of the chair that you want to assign to a Preset Position key.

2 Press and hold “0” or “1” for five seconds. A long audible beep will sound to indicate that the assignment is complete.

OPERATION - HEADREST

DUAL ARTICULATING MECHANISM

Reposition the headrest by first supporting the Headrest Bar (right image, item 2) with one hand and then pressing and holding the Headrest Button (right image, item 1) with the other hand to release the lock. Once the lock has been released, both pivot points will rotate freely and allow for adjusting both the elevation and the tilt of the headrest. Release the Headrest Button to lock the new headrest position.

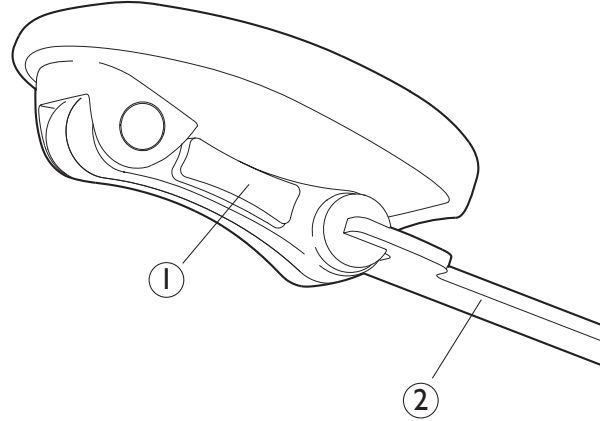

HEADREST BAR

REPOSITIONING

Reposition the height of the headrest so that it aligns to the patient's head by lifting up on the headrest. The tensioning mechanism on the headrest bar will allow the headrest to hold in position. Do not remove the headrest beyond the marked line on the headrest bar when a patient is seated.




WARNING: Removal of the headrest past the printed line could result in injury to the doctor or patient.

WARNING:
DO NOT REMOVE
PAST THIS LINE

TENSION ADJUSTMENT


Increase or decrease the tension on the headrest bar by using a hex wrench to tighten or loosen the Tension Adjustment Screw (below, item 1) beside the opening for the headrest bar at top of the backrest.

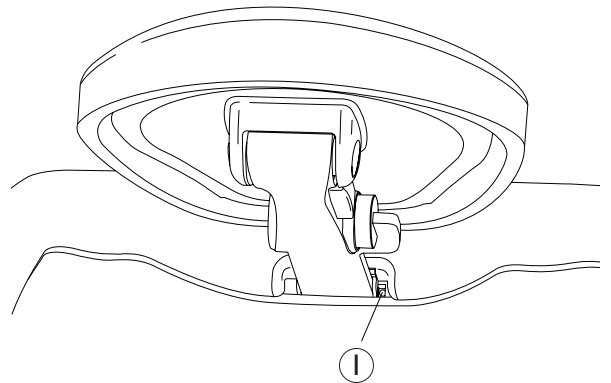

OPERATION - SWIVEL


SWIVEL ROTATION

The upper half of the chair swivels 30 degrees to each side of the centerline of the chair for the purpose of maximizing space in the operatory and gaining better access to the oral cavity.

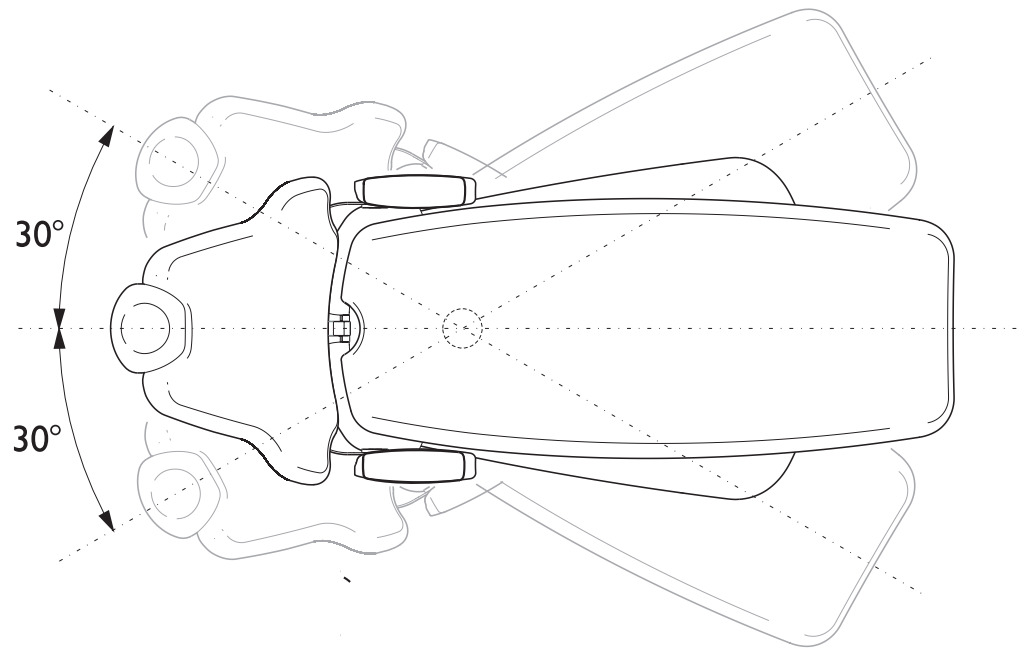

SWIVEL BRAKE HANDLE


Rotate the upper half of the chair by releasing the swivel brake handle accessed at the rear of the chair. Lock the position by returning the handle to the original position.

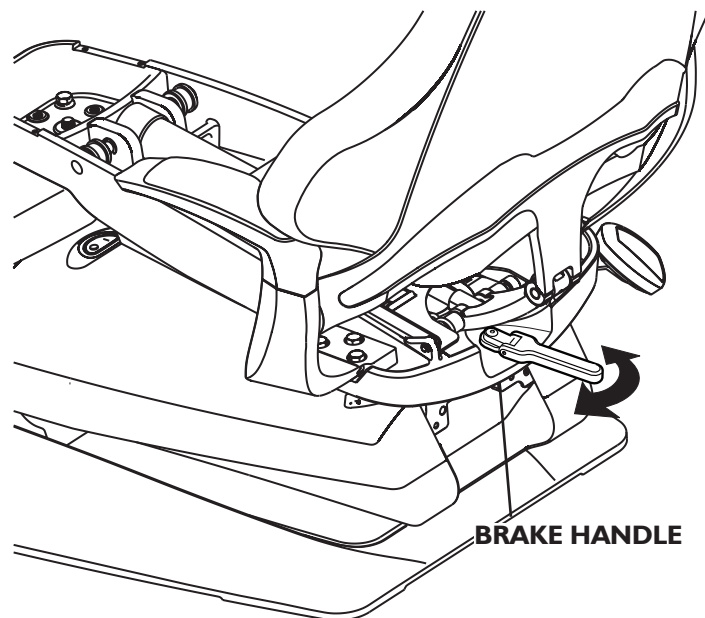

OPERATION - SAFETY STOPS

SAFETY STOP


Downward motion is stopped in the event that the underside of the truss impacts an obstruction during its downward travel. The Safety Cover located on the bottom side of the truss is designed to shift and engage safety switches located on the either side of the truss. While the safety switches are engaged, downward travel of the chair is disabled for the purpose of safety.

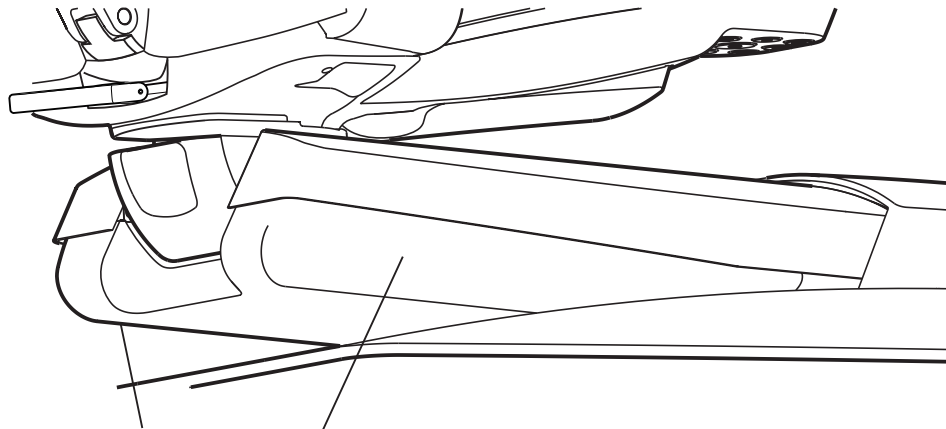

SAFETY SWITCHES BEHIND COVER

FOOT CONTROL ACTIVATED LOCKOUT

When configured in combination with a Series 5 dental delivery system, chair movement is disabled when the foot control is pressed. When the foot control is not actively being pressed, chair movement returns to normal functionality.

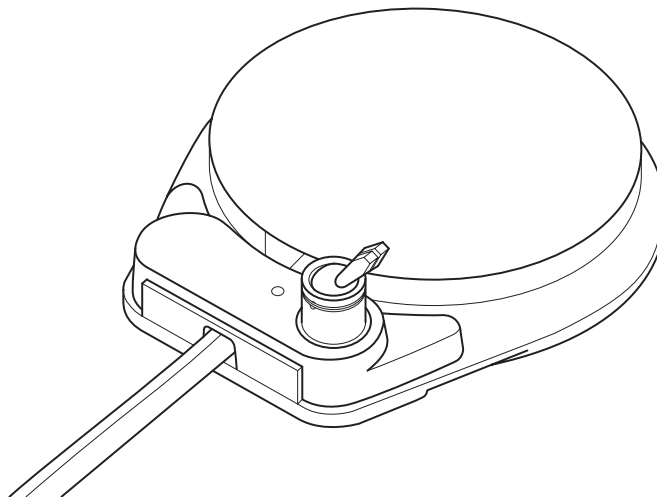

OPTIONAL QUICK CONNECTION BOX


ELECTRICAL OUTLETS

The optional quick connection box is conveniently located underneath the seat, providing air and water outlets and a duplex electrical outlet. These outlets can accommodate extra auxiliary equipment that is within the user's reach.

The electrical outlet is rated at a maximum of 1.5 amp per receptacle. If the 3 amp circuit breaker should open, reset it by pressing the reset button.



WARNING: Do not connect extension cords or power strips to the electrical receptacle's optional Quick Connection Box.

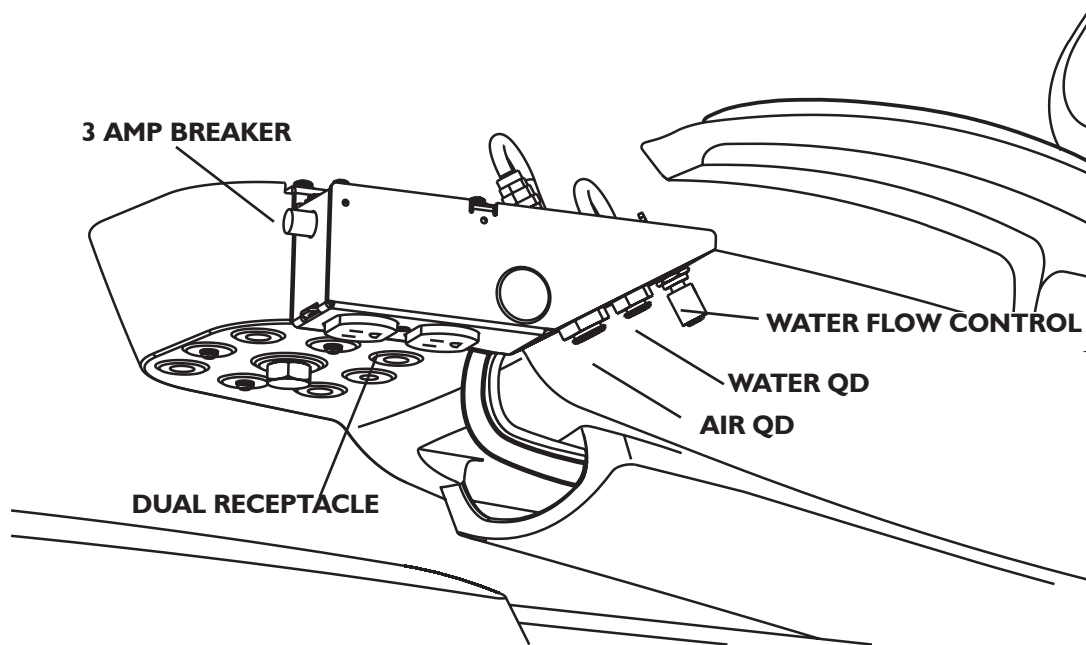

AIR AND WATER CONNECTIONS

The water outlet accepts a 1/4" QD fitting and has an integral shut-off valve. Next to the water outlet is a control valve to adjust flow from the water outlet. The air outlet accepts a 3/8" QD fitting and has an integral shut-off valve.

OPTIONAL AIRSOOTHE™ MASSAGE TECHNOLOGY


OVERVIEW

The AirSoothe™ Massage feature is installed on all C5552 model chairs. This feature is intended to create a soothing and calming effect for the patient and is powered by the compressed air in the office that is supplied to your chair.

HOW IT WORKS

Two sets of air bladders are integrated into the backrest upholstery. Each air bladder set inflates and deflates as compressed air is delivered through an integrated valve assembled to the backrest upholstery.

A pressure switch which is integrated into the upholstered seat signals to the chair to open the valve which controls the flow of air to the AirSoothe system.

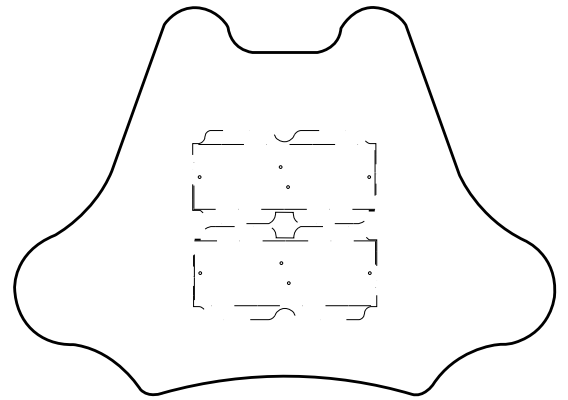


CONTROLS

Automatic Activation/Deactivation

- Activation occurs when a patient is seated in the chair. It will remain activated when the patient is reclined.
- Deactivation occurs when the patient leaves the chair after the backrest has been raised for exit. If a patient leaves the chair while the backrest is reclined, AirSoothe will remain on. AirSoothe will deactivate once the backrest is raised.

Manual Activation/Deactivation

AirSoothe will only function when the AirSoothe toggle is in the ON position. This toggle is located under the right Arm Support labeled with the AirSoothe logo. When the toggle is in the OFF position, air will not flow through the system even when a patient is seated in the chair.

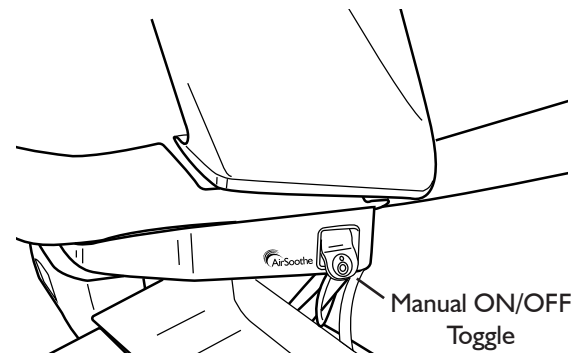

FACTORY SET

AirSoothe has been optimized to provide a soothing experience to the patient while minimizing airflow consumption. A built-in low pressure regulator is factory set and fixed to ensure a long-lasting and efficiently performing system.

MAINTENANCE

BARRIER TECHNIQUE

Wherever possible, use disposable barriers and change them between patients. The barrier technique will ensure maximum long term durability of the surfaces and finishes of the equipment.



NOTE: It is very important that all staff responsible for the maintenance, disinfection, and cleaning of this device are fully trained on the information provided in this section.

CLEANING

Dental equipment, categorized as noncritical devices with low risk of infection transmission, come into contact only with intact skin. Before applying disinfectant, clean the surfaces with a mild detergent and a clean cloth to remove unwanted dirt and debris.

CHEMICAL DISINFECTION

Follow the instructions below carefully to ensure the longest life for your equipment:

1. Only use the acceptable disinfectants listed. Use of unacceptable products may void your warranty.
2. When applying chemicals with a spray bottle, **do not spray surfaces directly**. Instead, spray a cloth and then wipe the surfaces with the wet cloth.
3. When using chemical disinfectants, always pay strict attention to the manufacturer's disinfectant directions.
4. When using concentrated disinfectants, measure the concentrate carefully and mix according to package directions. **Disinfectant solutions at higher than recommended dilution ratios are extremely corrosive.**
5. Thoroughly wash all areas that have been exposed to disinfectant cleaners with mild soap and warm water at least once per day. This wash down will minimize the harmful effects of chemical disinfectant residues being allowed to accumulate on the equipment.

Conditionally Acceptable Disinfectants

- Phthalaldehyde
- Quaternary Ammonium
- Glutaraldehyde



CAUTION: These disinfectants will harm the surface finish of dental equipment and are not recommended.

Unacceptable:

- Strong Phenols/Phenol Alcohol combinations
- Sodium Hypochlorite/Household Bleach
- Sodium Bromide
- Strong Alcohol
- Household Cleaners (Dental Equipment Only)
- Citric Acids
- Iodophors
- Ammonium Chloride
- Accelerated Hydrogen (0.5%)

MAINTENANCE

GENERAL CLEANING

Equipment can be cleaned with a solution of mild detergent and warm water. A variety of surface disinfectants are available for use in dental treatment rooms. Some of these can cause discoloration of painted, plated or anodized surfaces with repeated use. This can be minimized by careful adherence to the disinfectant manufacturer's instructions and by frequent washing with soap and water.



CAUTION: Do not use powdered cleansers, scouring pads or abrasive scrubbers on any of the painted, plastic or metal surfaces of this dental delivery system. To remove dried-on material, use a soft-bristled brush and a solution of mild detergent.



NOTE: Follow the manufacturer's recommended maintenance schedule, including regular inspections and servicing by qualified technicians.

CLEANING UPHOLSTERY



WARNING: In order to mitigate the risk of cross-contamination between patients, disinfection and sterilization of handpieces **must occur after each dental procedure**. Refer to the handpiece manufacturer's instructions and recommendations for sterilization or disinfection procedure.

NORMAL CLEANING

Mix a solution of 10 percent liquid dish soap with warm water and apply to the upholstery with a soft, damp cloth. Rinse with clean water and wipe dry. Cleaning frequency depends upon use. It is recommended that upholstery be cleaned between patients.

STAINS

Stubborn Stains

Use detergent cleaners such as Formula 409 or Fantastik. Wipe using a soft cloth or bristle brush. Rinse with clean water and wipe dry.

More Difficult Stains

Carefully clean the stained area with lighter fluid (naphtha) or rubbing alcohol. Apply with a soft white cloth and rub gently. Rinse with clean water and wipe dry.

OTHER TIPS

Always apply cleaners with a soft white cloth. Avoid the use of paper towels. When using strong cleaning solutions such as alcohol, it is advisable to first test in an inconspicuous area. Never use harsh solvents or cleaners that are intended for industrial use. To restore luster, a light coat of spray furniture wax may be used. Apply to chair and allow to set for 30 seconds. Lightly buff dry with a clean, dry cloth.

ELECTROMAGNETIC COMPATIBILITY

DEVICE COMPATIBILITY

ELECTRICAL MEDICAL

Electrical medical devices are subject to special EMC safety measurements and as a result the equipment must be installed according to the installation instruction manual.

PORTABLE ELECTRONIC DEVICES

Portable and mobile high frequency electronic communications equipment may interfere with electronic medical devices.

STATIC SENSITIVE DEVICES

Where labeled this equipment contains static sensitive devices that require special precautions when handling.

At a minimum, a grounded wrist strap that is connected to a ground stud should be worn to reduce the possibility of damage.

ACCESSORY USE

Using accessory devices not specified by DCI for use with their equipment may result in an increase of electromagnetic emissions and/or a decrease in electromagnetic immunity of the system.

INTERFERENCE FROM OTHER EQUIPMENT

If other equipment is used adjacent to or stacked with the DCI Edge equipment, the system must be observed to verify normal operation.

DCI EDGE EQUIPMENT COMPLIANCE

This equipment has been tested and found to comply with the limits for medical devices in IEC 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. In the event of interference, power the devices from separate mains supplies and/or increase the physical distance between devices. Contact Customer Service if you have any questions.

BLUETOOTH

FCC

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

ISED

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

Cet appareil est conforme à la norme sur l'innovation, la science et le développement économique (ISED) norme RSS exempts de licence. L'exploitation est autorisée aux deux conditions suivantes: (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

